


philips onsite aed guide

1. Call 9-1-1, send someone to get the AED, start chest compressions
2. When AED arrives, un-zip the red case, push the green ON-button

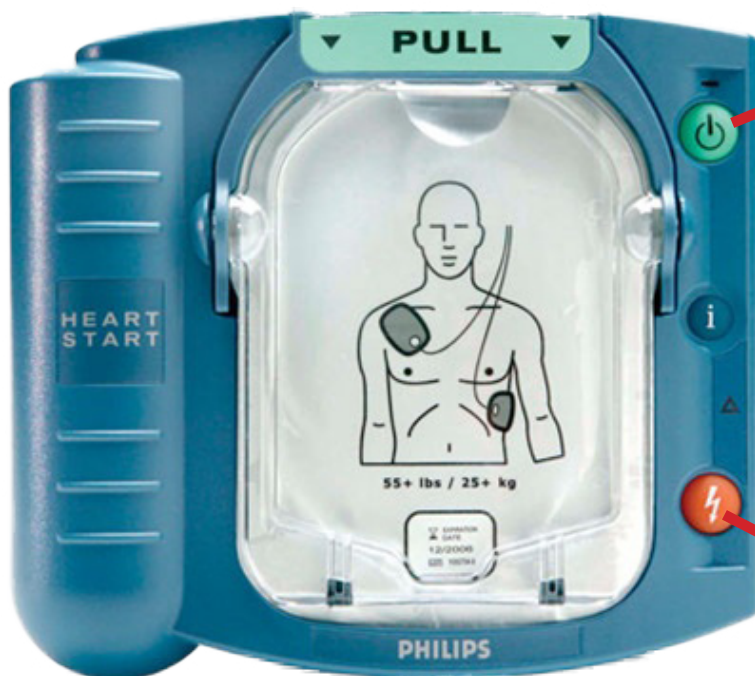

3. Pull down on the cartridge to expose the AED pads. Follow the pictures on the pads and apply directly to patient's chest.

4. Once pads are attached to chest, allow machine to analyze. It will tell you if a shock is needed. If so, orange button will start to flash. Make sure no one is touching the patient, press orange button.
5. After shock is given, begin chest compressions again. For reminders, press the flashing blue button.